Endofill

Root canal filling material with Dexamethasone

Radiopaque paste for the permanent obturation of the root canal

Composition

Powder Thymol Iodide 22.5%, Polyoxymethylene 2.2%, Hydrocortisone Acetate 1.0%, Dexamethasone Acetate 0.01 %, excipient ad 100%

Liquid Eugenol

Properties

Endofill is a radiopaque preparation for permanent root canal filling. Its composition is well tolerated by tissues and it provides anti-inflammatory, antiseptic and germicidal actions. Before hardening, the paste penetrates the narrowest fissures and maintains its therapeutic effects throughout the treatment until completely set. The final obturation neither retracts nor resorbs.

Indications

Permanent obturation of the root canal either after pulpectomy or after treatment of infected 4th degree cavities, preferably associated with endodontic points.

Directions for use

- ▶ Clean the canal thoroughly and dry with hot air
- ▶ Mix the powder and liquid until to obtain a paste of the required consistency.
 - **Endofill Regular**: the recommended powder/liquid (P/L) weight ratio is between 2.3 and 3.5 (g/g)
 - **Endofill Slow**: the recommended powder/liquid (P/L) weight ratio is between 2.5 and 3.1 (g/g)
- ▶ Work the paste into the root canal with the aid of a probe or a rotary paste filler. Fill the canal completely, avoid air pockets, do not protrude beyond the apex
 - **Endofill Regular**: the setting time (tested according to ISO 6876) is 80-90 minutes, depending on the P/L ratio (the thicker the paste, the faster is the setting time)
 - **Endofill Slow**: the setting time (tested according to ISO 6876) is 17-24 hours, depending on the P/L ratio (the thicker the paste, the faster is the setting time)

Warning

Contains Polyoxymethylene which can be toxic by inhalation or ingestion. When using this product, gloves, mask and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

Storage

Store at room temperature between 5°C and 30°C, protected from direct light and humidity.

Shelf-life 3 years.

Packaging

| | REG regular | REF slow |
|---|--------------------|-----------------|
| Portion 15g powder + 15ml liquid | 10270 | 10275 |
| Bottle of 15g powder | 10271 | 10276 |
| Bottle of 15ml liquid | 10272 | 10272 |

For dental professional use. Made in Switzerland.

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© PD 160517