Endoseptone

Camphor Chlorophenol Thymol with Dexamethasone. Solution for root canal disinfection.

Composition

Camphor 64.9%, p-Chlorophenol 30%, Thymol 5%, Dexamethasone Acetate 0.1 %

Properties

Endoseptone is a solution which offers a triple-action; bactericidal, sedative and anti-inflammatory, thanks to the added Dexamethasone. It also reduces the risk of allergic reactions.

Indications

- Disinfection of the root canal and deep caries
- Treatment against gangrene of the pulp
- Disinfection of narrow, obturated or partially calcified root canals

Directions for use

Dry the root canal thoroughly

- **Following a pulpectomy** insert 1 or 2 drops of the preparation into the root canal and allow a few minutes for it to take effect. Remove any excess solution with a cotton pellet or a gauze
- **Following the extraction of gangrenous pulp** use as a dressing by inserting into the root canal with a smooth broach. Leave «in situ» for at least 3 days. Make sure the canal is completely disinfected after that period. If this is not the case, repeat the dressing before the final obturation

Warning

If used after an arsenical treatment, first rinse the root canal with an aqueous iodine solution, then rinse with water and dry. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

Storage

Store at room temperature between 15°C and 25°C, protected from direct light.

Shelf life

3 years.

Packaging

Bottle of 15 ml

REF
11350

For dental professional use. Made in Switzerland.

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© PD 160712