DIRECTIONS FOR USE

GB

Iodoform Gauze

Dressing for the treatment of extraction sockets

Composition

5% Iodoform Gauze 100 g of gauze impregnated with 5 g of Iodoform
10% Iodoform Gauze 100 g of gauze impregnated with 10 g of Iodoform

Properties

Iodoform Gauze is a mild analgesic which acts as an antiseptic agent. When reacting with wound secretions and fatty acids, the Iodoform frees Iodine to generate, by combination with proteinic substances, an active germicide.

Indications

- Prevention of periodontal infections
- Post extraction disinfecting dressing
- Disinfection of maxillary cavities

Directions for use

Cut the necessary length of gauze to fit the extraction socket and pack gently into the cavity to be treated.
The gauze is non-resorbable and can remain in situ for 2 to 8 days prior to removal at the end of the treatment.

Warning

When using this product, gloves and protective glasses should be worn. If skin irritation occurs (allergic reaction), get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing consult a doctor.

Storage

Store at room temperature between 15°C and 25°C, protected from direct light.

Shelf life 3 years.

Packaging

5% Iodoform 11480
10% Iodoform 11481

For dental professional use. Made in Switzerland.

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© PD 190822