Iodoform Paste

Antiseptic, resorbable and radiopaque paste for the temporary treatment of infected root canals

Composition

Iodoform 75%, Camphor 8%, Chlorophenol 6%, Menthol 6%, excipient ad 100%

Properties

Iodoform Paste is a well-known preparation for the treatment of infected root canals as well as for deep-seated gingival pockets. The components of the paste ensure a long-lasting germicidal and analgesic action. Furthermore, the paste does not harden and is easily removable mechanically which also facilitates its apical resorption.

Indications

- Infected root canals, including those of deciduous teeth
- Deep-seated gingival pockets

Directions for use

Infected root canal. Remove the pulp, clean, rinse and dry the root canal. Work the paste into the canal with a smooth broach or a rotary filler. Avoid air-pockets or piercing the apex, except in the case of granuloma with fistula. Protect with a temporary filling. 20 to 30 days after the treatment, check whether the symptoms have settled and, if so, clean the canal carefully, making sure that no residual material flows back through the apex. Check the aseptic of the canal, if it is not proved, treat again. Obtrurate permanently and complete the restoration. Infected, deep-seated gingival pockets: Place the paste onto a gauze and use it as an alveolar dressing which can be left «in situ» for 2 to 3 days. Remove the gauze once the symptoms have disappeared.

Precautions

Close the jar tightly after use. If the paste should harden, use Eugenol to smoothen it. Occasional cases of staining of the tooth structure have been reported following the use of treatment preparations containing Iodoform. Hypersensitivity has also been reported. Iodoform Paste should be used accordingly.

Warning

In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

Storage

Store at room temperature between 5°C and 30°C. Protected from direct light.

Shelf life

3 years.

Packaging

Jar of 15 g

For dental professional use. Made in Switzerland.

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© PD 140512