Surgical Dressing

Alveolar analgesic and antiseptic dressing for dental use

Composition

Guaïacol 12.2%, Benzocaïne 8.2%, Chlorobutanol 5.1%, Menthol 5%, Thymol iodide 4.1%, Methyl Salicylate 1.8%, excipient ad 100%

Properties

Surgical Dressing is a paste of creamy consistency to be used for treating post-operative dental pains and to combat alveolar infection. The dressing provides almost instant pain relief. Its action is long-lasting as the paste is completely absorbed by the tissues.

Indications

Post-operative pain following tooth extraction and all other dento-surgical operations. Relief of pain in gingival tissues and during the treatment of periodontal packets. Dry, exposed or denuded alveolar sockets. Lacerated or traumatized tissues as well as similar wounds where dental pain and infections are to be feared

Directions for use

- Apply the necessary quantity of paste to the area requiring treatment
- Where necessary, the application can be repeated during 7 days

Precautions

Avoid using Surgical Dressing for patients who present allergic reactions to Benzocaïne or try any other component or in case of congenital or idiopathic methemoglobinemia. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water and consult a doctor. Keep the person exposed at rest, do not force vomiting.

Storage

Store at room temperature, between 5° and 30°C, protected from direct light.

Shelf life

5 years.

Packaging

Tube of 25 g11630

For dental professional use. Made in Switzerland.

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© PD 180920