Tricresol & Formalin

Solution for root canal disinfection

Composition

Tricresol 35%, Formaldehyde (40%) 19%, excipient ad 100%

Properties

Since its early introduction in 1905 by Buckley, Formocresol, thanks to its antibacterial activity, has proved as efficient in vital pulpotomy as for root canal treatment.

The Formaldehyde acts as a powerful, long-lasting germicide and also renders inert any remaining organic materials. The Cresol acts as an efficient antiseptic and also reduces any irritation that could be caused by the Formaldehyde.

Indications

- Treatment of 4th degree caries
- Root canal dressing, specially when fistula have developed

Directions for use

Clean and prepare the canal. Dip a cotton gauze into the solution, squeeze until no longer dripping and insert into the canal. Seal with a temporary cement. Remove the temporary filling and the gauze after 3 to 5 days and check the asepis of the canal. Where necessary, the treatment can be renewed. Close cavity with a permanent filling.

Warning

Contains Formaldehyde. Toxic by inhalation and ingestion. May cause irritation, burns or hypersensitization if in direct contact with the skin. In case of direct contact with the eyes, rinse liberally with running water and consult a physician.

Storage

Store at room temperature between 5°C and 30°C, protected from direct light.

Shelf-life

3 years.

Packaging

Bottle of 15 ml	11373
Bottle of 30 ml	11372

For dental professional use. Made in Switzerland.

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© PD 140512